

This is your smartheart quick start guide

First Time Set Up



for more information go to:

Pairing guide section A

Pairing guide section B

User guide section 2.1

Pairing guide section C

1 Activate your account


Click on the link sent to your email
by support@smartheartpro.com

2 Download the app

Download the Smartheart app
from the App store or Google Play.
You can find the app by typing
Smartheart in the search box.



3 Insert the batteries

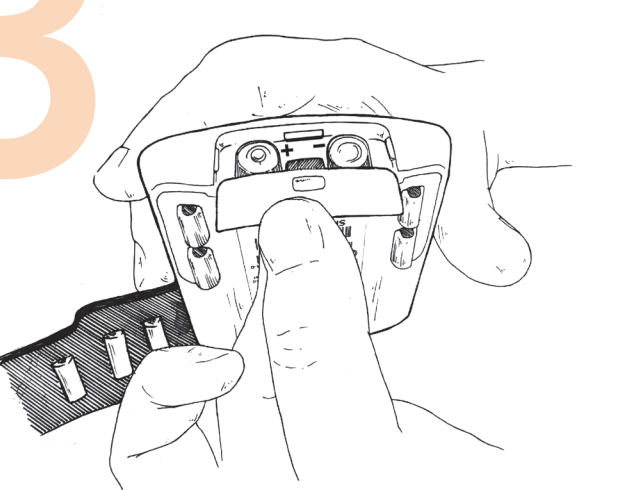

4 Pair your Smartheart


Launch the application to view
pairing Instructions.
The requested PIN code is the serial
Number on the label of the device.

The process may take up to 2
minutes.



5 Log into the Application

To Login use your email and the
password you have chosen while
activating your account.

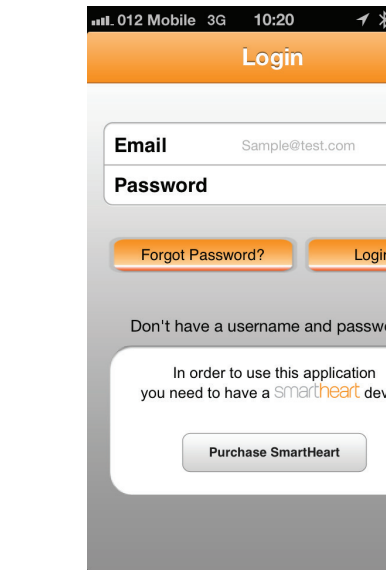

Go to www.SmartheartPro.com
for additional information.

This page does not replace the
User Guide, it is recommended
to read the user guide before
using the Smartheart




Performing an ECG



for more information go to:

1 Moisten the Body

Moisten with water body parts that need to come in contact with the electrodes (left & right arm inner parts, waist and chest).

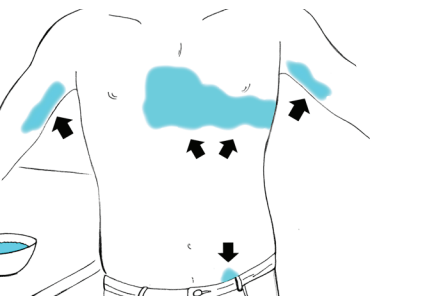

User guide section 3

2 Place the Smartheart


Place the Smartheart against the center of the chest. The belt should be:

- Male: Approximately one centimeter (1/2 inch) under the nipples.
- Female: Under the breasts, as high as possible.

Tighten the belt. Position the right arm, left arm & waist electrodes.




User guide section 3

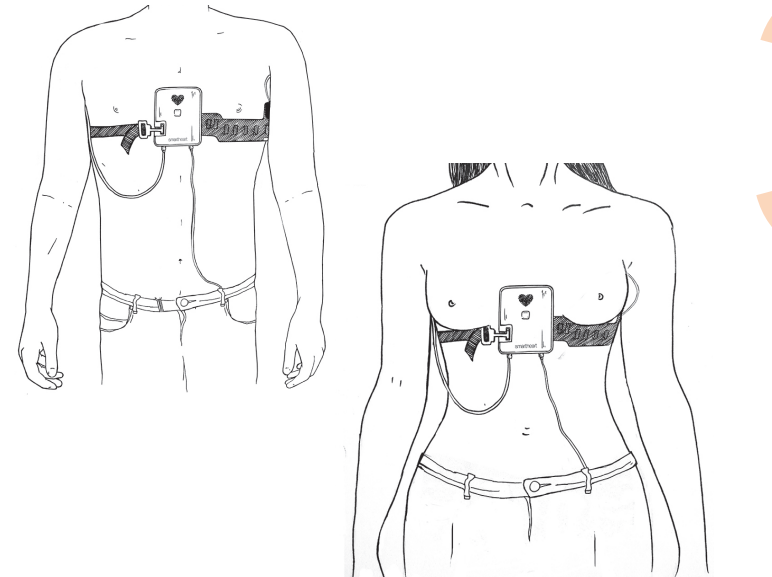

3 Identifying the chest size

Drawing an imaginary line between the middle of the left underarm and the left hipbone. Connection point indicates the chest size

Note: Gender and size will be required by the application every time a new profile is created.

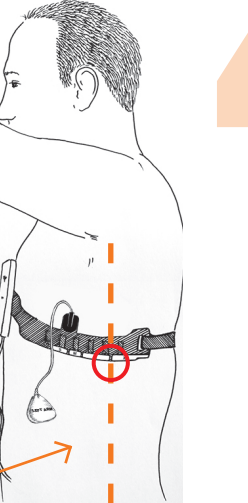

User guide section 2.3

4 Launch the app

Tap perform ECG.




Select an existing profile or create a new one for your patient.

Follow the instructions in the application.

App Help

5 Sit comfortably

ECG is performed while sitting down, leaning slightly and placing both hands between the thighs.

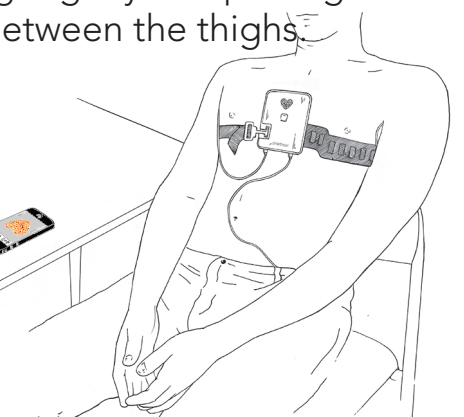

User guide section 3

6 Verify Electrodes contact

Be sure that all the electrodes are in contact with the body and shown in green. Tap the START ECG button.


Relax muscles, lean back and stay still until the ECG is done

App Help

7 View ECG

Follow instructions prompted by the app to view feedback or forward the ECG for review.

When this link appears, tap it for instructions on how to technically perform your ECG better →

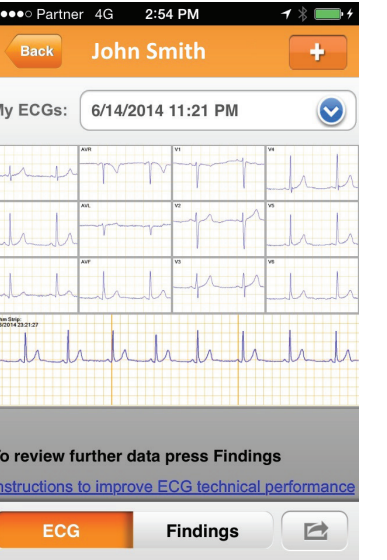

App Help